

Protective Face Shield Visor – Reference 2005-Eco Specification Sheet



Our Ref 2005-Eco is an eco-friendly biodegradable protective face visor. It is a lightweight, clear face shield for personal use, that will create a barrier to help reduce droplets and sneezes.

Here at UK Industrial Tapes we are fully committed to eco-friendly biodegradable* plastic products so we are very excited to be working with Breakdown Plastics Inc to bring you our new Eco-Friendly Range.

USP

- The Protective Face Shield Visor – Reference 2005 is lightweight in use and easy to wear
- It has a Green Injection moulded Polyprop adjustable zip effect head strap
- Our Green Injection moulded head strap is very strong and durable and can be re-used every day
- The Face Shield can be cleaned with warm water and any kind of cleaning detergent
- Our visors are produced from Eco Friendly clear recycled APET.

Cleaning, Disinfection and re - use

- Our 2005 face shield can be cleaned with warm water and any kind of cleaning detergent
- Our Green Injection Moulded head strap is very strong and durable and can be sanitized and cleaned every day ready for re use on a daily basis

Application

- Personal use
- Help support essential business and NHS staff

Product Description

- A lightweight, clear face shield for personal use, that will create a barrier to help reduce droplets and sneezes. The parts used are removable, cleanable and replaceable.
- Scratched or damaged visors should be replaced

Features

- Very lightweight material
- Practical minimal design
- Reduction of droplets from coughs and sneezes

Advantages

- Extremely lightweight material
- Easily cleaned
- Durable Re-usable green Injection Moulded head strap
- Long service life use

Breakdown Plastic Ince – The Science

- Change Plastic for Good® offers a range of biodegradation-enhancing polymers and renewable resins that apply to a specific end-of-life scenario for plastic, rubber, foam, fabric and coated paper waste.

Breakdown Plastic Ince – In short

- Ingredients in BDP™ accelerate the biodegradation of otherwise non-biodegradable plastics by providing a foothold for colonies of microorganisms. This foothold allows the microorganisms to begin metabolizing the treated plastics.
BDP™ can be used in most plastics, synthetic fabric and yarn, rubber and foam

Storage Conditions

- When stored in stacks, protect against heat, moisture and direct sunlight

Use & Handling

- Instructions on how to prepare and use the Face Shield have been provided
- Place your hand on the inside when putting on the Face Shield to avoid getting scratched by the edges

Period of obsolescence

- Our 2005 is a single use product only to be used for no more than 12 hours
- Replacement packs of 50 clear face shields can be purchased

T: (0191) 269 7810 | F: (0191) 267 1028

E: sales@ukindustrialtapes.co.uk | sales@postapes.co.uk | sales@stix2.co.uk

Brumwell House, Westway Industrial Estate, Throckley, Newcastle Upon Tyne, NE15 9EW

www.ukindustrialtapes.co.uk | www.postapes.co.uk | www.stix2.co.uk



It is the buyer's responsibility to consider & assess the fitness for purpose of the goods, having regard to use to which the buyer intends to put such goods. The price is quoted for the quantity quoted. All prices quoted are for 30 days only. Standard credit terms are 30 days from the end of month unless agreed in writing otherwise. All orders are subject to this Company's terms & conditions of trading which can be seen on our website ukindustrialtapes.co.uk, postapes.co.uk and stix2.co.uk



POS Tapes & Stix2 are trading divisions to UK Industrial Tapes Ltd

Cautions

- Do not apply much pressure when bending the Face Shield to avoid breaking it
- A warning that materials which may come into contact with the wearers skin could cause allergic reactions to susceptible individuals
- Eye-protectors against high speed particles worn over standard ophthalmic spectacles may transmit impacts, thus creating a hazard to the wearer

Packaging

- They are packed in a self-adhesive FDA food approved OPP bag. We pack our visors, straps, specification sheet and assembly instructions to customers requirements.
- Shipped in standard shipping carton or jiffy bag depending upon quantity ordered

Replacement face Shields

- We supply a pack of 50 replacement face shields which can be easily replaced onto our head strap after every shift

Product Material

- Protective Face Shield Visor – Reference 2005 (210mm x 298mm)
- 275 clear Micron Recycled APET
- Green Injection moulded Polyprop adjustable zip tie effect head strap

***Disclaimer: Visor 2005 does not have a CE mark therefore customers purchase at their own risk UK Industrial Tapes Limited will not accept any liability.**

***Disclaimer: Whilst our Protective Face Shield Visor - Reference 2005 has many applications it is not designed to be used with Covid-19 positive patients on its own without the use of other forms of PPE.**

T: (0191) 269 7810 | F: (0191) 267 1028

E: sales@ukindustrialtapes.co.uk | sales@postapes.co.uk | sales@stix2.co.uk

Brumwell House, Westway Industrial Estate, Throckley, Newcastle Upon Tyne, NE15 9EW

www.ukindustrialtapes.co.uk | www.postapes.co.uk | www.stix2.co.uk



It is the buyer's responsibility to consider & assess the fitness for purpose of the goods, having regard to use to which the buyer intends to put such goods. The price is quoted for the quantity quoted. All prices quoted are for 30 days only. Standard credit terms are 30 days from the end of month unless agreed in writing otherwise. All orders are subject to this Company's terms & conditions of trading which can be seen on our website ukindustrialtapes.co.uk, postapes.co.uk and stix2.co.uk



POS Tapes & Stix2 are trading divisions to UK Industrial Tapes Ltd