

Protective Face Shield Visor – Reference 2005-Eco

Visor Assembly Instructions



in partnership with



**Change
plastic
for good**®



T: (0191) 269 7810 | F: (0191) 267 1028

E: sales@ukindustrialtapes.co.uk | sales@postapes.co.uk | sales@stix2.co.uk

Brumwell House, Westway Industrial Estate, Throckley, Newcastle Upon Tyne. NE15 9EW

www.ukindustrialtapes.co.uk | www.postapes.co.uk | www.stix2.co.uk




Step 1.

Have the Green Visor and the Clear APET Sheet at the ready.



Step 2.

- Attach the left hole on the clear APET sheet into the left bracket point, with the green visor facing you.
- Attach the 2nd hole on the clear APET sheet into the 2nd bracket point on the green visor.
- Attach the 3rd hole on the clear APET sheet into the 3rd bracket point on the green visor.
- Attach the 4th hole (only hole with a small slit on the right side, this is to ensure the hole can fit into the bracket point with ease) into the 4th and last bracket point with slight pressure.

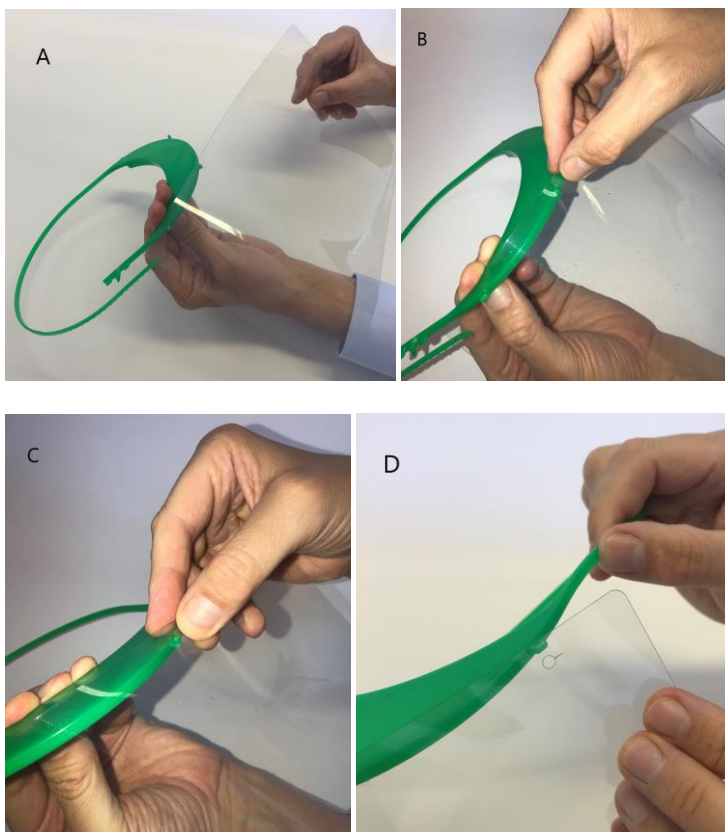

T: (0191) 269 7810 | F: (0191) 267 1028

E: sales@ukindustrialtapes.co.uk | sales@postapes.co.uk | sales@stix2.co.uk

Brumwell House, Westway Industrial Estate, Throckley, Newcastle Upon Tyne. NE15 9EW

www.ukindustrialtapes.co.uk | www.postapes.co.uk | www.stix2.co.uk



It is the buyer's responsibility to consider & assess the fitness for purpose of the goods, having regard to use to which the buyer intends to put such goods. The price is quoted for the quantity quoted. All prices quoted are for 30 days only. Standard credit terms are 30 days from the end of month unless agreed in writing otherwise. All orders are subject to this Company's terms & conditions of trading which can be seen on our website ukindustrialtapes.co.uk, postapes.co.uk and stix2.co.uk



POS Tapes & Stix2 are trading divisions to UK Industrial Tapes Ltd

Step 3.

To fix the head strap to suit your preferred size, insert the tie through the small hole on the green visor. The size can be adjusted with the 18 size clips to suit the preferred fitting. To re adjust the size, you can feed the tie back through the hole.



Step 4.

The visor can then be placed onto the head and secured in a comfortable position for your personal protection.



Any further questions you may have regarding our Protective Face Shield Visor – Reference 2005-Eco, please contact our sales team right away.

***Disclaimer: Visor 2005-Eco does not have a CE mark therefore customers purchase at their own risk UK Industrial Tapes Limited will not accept any liability.**

***Disclaimer: Whilst our Protective Face Shield Visor - Reference 2005-Eco has many applications it is not designed to be used with Covid-19 positive patients on its own without the use of other forms of PPE.**

T: (0191) 269 7810 | F: (0191) 267 1028

E: sales@ukindustrialtapes.co.uk | sales@postapes.co.uk | sales@stix2.co.uk

Brumwell House, Westway Industrial Estate, Throckley, Newcastle Upon Tyne. NE15 9EW

www.ukindustrialtapes.co.uk | www.postapes.co.uk | www.stix2.co.uk



It is the buyer's responsibility to consider & assess the fitness for purpose of the goods, having regard to use to which the buyer intends to put such goods. The price is quoted for the quantity quoted. All prices quoted are for 30 days only. Standard credit terms are 30 days from the end of month unless agreed in writing otherwise. All orders are subject to this Company's terms & conditions of trading which can be seen on our website ukindustrialtapes.co.uk, postapes.co.uk and stix2.co.uk



POS Tapes & Stix2 are trading divisions to UK Industrial Tapes Ltd